#rifton Compass Chair





Compass Chair

It's the perfect inclusion chair. The rock-solid frame is built to withstand years of demanding use. The gracefully contoured seat and sturdy armrests provide reassuring boundaries, giving mildly involved students the support they need to feel secure and comfortable. At the same time, its simple and elegant construction lets it blend easily into any classroom environment. It looks almost like a regular classroom chair – it's meant to.

The five available sizes accommodate everyone from small children to adults, and the adjustable legs allow fine-tuning for optimal positioning.



Find important details and informative facts about the Compass Chair at: www.rifton.com/compasschair

We have had trouble getting one of our students to sit even for a few moments, which, of course, makes learning very difficult.

We've tried different seating options but nothing seemed to work. Imagine our surprise when he saw the Compass Chair for the first time and, entirely on his own, went and sat in it until we had to ask him to get up!

Kathleen Marshall Anderson Center for Autism Staatsburg, NY



Compass Chair features





Unique built-in support
The contoured sides of the seat, together with the inside surfaces of the armrests, provide subtle lateral support that can be particularly helpful for students with Autism Spectrum Disorder, giving security and sensory boundaries.



The Compass Chair comes in five sizes.





Stability feet attach to the back legs to accommodate your more active, tip-prone students.



Adjusting the rear legs shorter than the front legs gives the chair a slight tilt, which can help some students relax.

Components



Seat belt

The optional seat belt clips into the seat when needed for safety or postural support.





Stability feet

The stability feet enlarge the chair's base of support. They are adjustable like the standard chair legs.





Adjustable legs

The legs provide two inches of height adjustment to ensure optimal seat height and allow for growth. Both chairs in this picture are size 3.



Compass Chair dimensions

User dimensions (inches)	R310 size 1	R320 size 2	R330 size 3	R340 size 4	R350 size 5				
Height	36 – 43	43-48	48-55	55-63	63+				
Knee to heel	9-11	101/2-121/2	121/2-141/2	141/2-161/2	161/2-181/2				
Key user dimension: height The user's overall height is a general guide to help you select the appropriate chair. Choose the model that allows for growth.									
Important: Make sure that seat width, depth, and height are adequate for each user, and that the user's weight does not exceed the maximum weight recommended.									
Item dimensions (inches)	R310	R320	R330	R340	R350				
Seat height	9,10,11	10½,11½,12½	12½,13½,14½	14½,15½,16½	16½, 17½, 18½				
Seat depth	91/2	11	12¾	14¾	161/2				
Seat width	101/2	121/4	14	161/4	181/4				
Backrest height	91/2	11	121/2	141/4	16				
Armrest height above the seat	5½	6	7	81/4	8¾				
Armrest length	61/2	71/2	81/2	9¾	11				
Max. working load (lbs)	100	120	175	220	250				

Order form

Use dimension chart to select appropriate size.

	R310 size 1	R320 size 2	R330 size 3	R340 size 4	R350 size 5
*Compass chair	R311 🔲	R321 🗆	R331 🗆	R341 🔲	R351 🗆
Seat belt	R312	R312	R332	R342	R342 🔲
Stability feet	R316 🔲	R316	R316	R316	R316 🔲

^{*} Required



You asked; we delivered.



The return of an old friend

For years you've been asking for a simple supportive chair with armrests — like the old Compass Chair. Studying your suggestions and collaborating with therapists and teachers around the world, our team worked to develop the perfect (we hope!) inclusion chair.

While we kept the name, this is more than the reintroduction of an old product. The new chair has height adjustment, improved ergonomics and better support. We think it's the perfect chair for learning. Tell us if we got it right.