



Rifton Wave (Z220 with tub transfer base Z228)

The Wave

A Bathing & Transfer System

Hygiene and health go together. When we make bathing easier, our clients are healthier. The Wave system has been thoroughly reimagined to improve bathing and showering for the client and for the caregiver.

Responding to requests from therapists and caregivers, we widened the seat to accommodate larger clients. And to protect skin integrity, we added generous padding under soft knit fabric.

In addition to the tub stand and rolling shower stand, we've added an ingenious new tub transfer base to completely change the way you transfer and bathe large clients. There's nothing like it.



Find letters of medical necessity and informative articles at:
www.rifton.com/wave



“ The Wave bath chair is terrific! In the past, lifting a child into a tub has been a back killer, but the new transfer base eliminates this hazard entirely. The casters allowing lateral movement in the tub are a welcome addition, and the fact that I can easily collapse the base and hang it on the wall is a fabulous space saver. ”

Linda K. Miller, DPT
Queensbury, New York

Unique Features & Benefits

Include these in your letters of medical necessity:

- **Extra wide seat** with open sides, 15" width for small and medium, 17½" for the large.
- **Large size accommodates adults** up to 6'2" and 250 lbs.
- **Completely configurable;** all sizes of the Rifton Wave fit on any of the 3 base options.
- **Optional calf rest** adjusts independently of seat angle.
- **Tub transfer base** enables users—after being placed in the bath chair using a mechanical lift—to be rolled back and then rotated over the tub for showering.
- **Conversion kit** allows user to economically move up from small to medium size by just purchasing a new back, seat and cover.
- **Least costly alternative** to remodeling a bath room or assisted living, without resorting to bed baths.

Features of the Wave

No tools required



Generously sized

The wide seat and open design enables large users to fit comfortably. Seat width on the large size is 17½".



One-hand adjustment

Adjust backrest angle with one hand, keeping the other hand on the client for safety and reassurance.



Full leg support

The optional calf rest provides full support for the lower legs. Adjust it independently of the seat by pushing the gray buttons and rotating up or down. It must be installed at the time of purchase.



Typical bath tub height range

- 18 in.
- 14 in.





Tub stand



Shower stand



Tub transfer base



Infinite adjustability

Backrest angle, seat angle, and calf rest angle adjust individually.



More seat heights

For an intermediate seat height, put the rear legs up and front legs down. To get the chair low in the tub, put the front and back legs up. Fold the chair completely flat for storage.



Required components

Bath chair size and color



Choice of chest strap



Without lateral positioning

The simple chest strap safely positions the user in the Wave bath chair. It attaches with hook and loop where needed along the full height of the back, and buckles securely behind.



With lateral positioning

Rifton designed this chest strap with adjustable lateral straps to give additional lateral support helping users maintain better trunk alignment in the bath chair. The soft fabric laterals will not get in the way during transfers.



Accessories



Head blocks

Optional head blocks provide lateral support and positioning for the user's head.



Leg straps

Use the fully adjustable leg straps to position your client's hips in abduction (or just to provide additional security).



Calf rest

The new calf rest option provides more support for the lower legs and is especially needed when the bath chair is used on one of the stands.

Note: Calf rest must be chosen at the time of the original purchase. It cannot be added later.



Tub stand

The optional tub stand raises the bath chair either 5½" or 11½" higher in the tub depending on the position of the tub stand legs.



Shower stand

The shower stand lets you roll your client into the shower stall at a height that allows for better cleaning. Better cleaning means better health. Locking casters (4") provide smooth, easy maneuverability.



Accessories (continued)

Tub transfer base

Bathing equipment for people with disabilities tends to be big and bulky, taking up far too much space in bathrooms shared by the whole family. Now using Rifton's ingenious new tub transfer base you can perform healthy transfers without crowding your bathroom with permanently mounted equipment. When bath time is over the base collapses and hangs compactly on the wall.



Unique two-stage movement



The tub transfer base starts in the position farthest out of the tub. It is then rolled back over the tub, and the brakes are set.



As the upper portion with the bath chair is rotated 90° it moves approximately 12" back over the tub in one motion.



The tub transfer base fits bath tubs up to 24" high by adjusting the leg length to clear the edge of the tub.



1. With the client secured in the chair, push the transfer base to the back of the tub as far as it will go and lock the wheels.



2. Press either of the gray pivot latches and rotate the bath chair and client over the tub.



3. As the bath chair rotates, it continues to move back over the tub. Gently lift the client's legs to clear the edge of the tub.



4. In the final position the bath chair is over the tub and can be adjusted as desired for showering.


Free up the tub.

Rifton's folding, storable tub transfer base makes it possible for everyone in the family to still use the tub.

When you are finished, remove the base from the tub, fold the legs and hang it on the wall hook provided.



Dimensions of the Wave

User dimensions (inches)	Z210 small	Z220 medium	Z230 large
Height	up to 46	40–56	50–74
 Key user dimension: height Select the appropriate bath chair by the user's overall height. Choose the size that allows for growth. Important: Make sure that seat depth, width and back height are adequate. The user's weight must not exceed the maximum working load.			
	Item dimensions (inches)	Z210 small	Z220 medium
Seat depth	10½	13	17
Seat width	15	15	17½
Back height	23	26½	32
Seat height	2–7	2–7	2–7
Seat height (incl. tub stand)	12½–18½	12½–18½	12½–18½
Seat height (incl. shower stand)	27½	27½	27½
Seat height (incl. tub transfer base) with long leg inserts	21–26 24½–29½	21–26 24½–29½	21–26 24½–29½
Length of mesh area when flat (without calf rest)	35	41	50½
Length of mesh area when flat (with calf rest)	42½	49	61
Width of base	18	18	21
Length of base when flat	29½	29½	29½
Tub stand height (legs flat)	8	8	8
Tub stand length (legs flat)	39	39	39
Tub stand width (legs flat)	17½	17½	17½
Shower stand height	22	22	22
Shower stand width	27	27	27
Shower stand length	34	34	34
Tub transfer base footprint	28 x 30	28 x 30	28 x 30
Max tub height for transfer base with long leg inserts	20 24	20 24	20 24
Maximum working load (lbs)	75	150	250
Back and seat angle adjustments	0°, 15°, 30°, 45°, 60°, 75°		



Without calf rest
(Z269)

With calf rest
(Z267)

The conversion kit (two styles shown above) is the smart solution for growing children. Convert the small to a medium bath chair for a fraction of the cost of a new chair.

We'll send you a new larger fabric cover, back and seat frames and chest strap with instructions on how to remove the old and install the new parts on your existing base.



The Wave has a washable fabric cover that is easily removed for cleaning.



All three sizes are available in blue and pink.